# Carci Reagent<sup>®</sup>

**Test kit for the detection of an indicative amount of  
monohydroxyphenolic metabolites (tyrosine) in urine  
(Lay Semi-Quantitative Test for Self-Assessment)  
(Chemical chromogenic method)**

**Author: Mag. Martin Muzikant, Ph.D.  
Company: CNEU MEDICAL s.r.o.**

**Carci Reagent**

# The current situation in the prevention of oncological diseases

## SCREENING PROGRAMS OF HEALTH INSURANCE COMPANIES - Czech Republic

- ▶ Since 2014, addressed invitations for screening
- ▶ Over 8 million invitations in 4 years
- ▶ Aimed at breast, cervical and colon cancer screening
- ▶ Nevertheless, low interest of citizens and high non-participation

# The current situation in the prevention of oncological diseases

## Cancer prevention: The impact of the covid epidemic

- ▶ Half of people neglect colon and cervical cancer screenings  
approx. 50% of people do not go regularly, of which 20-25% at all!!!
- ▶ During covid, around 14% of people postponed key prevention, some did not go thanks to covid, up to 30% in the short term and approx. 15% in the long term did not participate in prevention
- ▶ CITIZENS CAN'T BE RETURNED TO REGULAR PREVENTION!

# Blood tests for the prevention of oncological diseases

## Tumor markers from blood and their price in the laboratory

- ▶ Currently, tumor markers are monitored either in blood or in tissue samples taken (biopsies)
- ▶ The total price of all available blood tumor markers is more than 400 USD, and this is only basic testing with variable sensitivity (60-80%).
- ▶ For example, the PSA test has 68-80% sensitivity, improved urine sediment test for PCA3 marker (Progenisa urine test) costs approx. 200 USD

# Basic examination in the framework of the prevention of oncological diseases I.

## Procedure in the prevention of oncological disease

- ▶ **Self-examination** - any findings on your body or doubts must be consulted with the treating general practitioner
- ▶ Patience when a malignant disease is suspected - unfortunately, at present, it is not possible to carry out all the examinations at once, it takes time.
- ▶ **Basic oncological examination** - blood and urine examination, imaging methods (SONO, Mammogram, X-ray, possibly CT), direct methods (gastroscopy, colonoscopy, bronchoscopy)

# Basic examination in the framework of the prevention of oncological diseases II.


## Basic examination prices for self-payers

- ▶ A basic examination for a possible oncological disease costs on average about 400 USD (self-pay)
- ▶ But if we are dealing with complex oncological examinations, then the prices are already over 1.200 USD (self-pay)
- ▶ And subsequent treatment of late-caught cancer costs hundreds of thousands USD

# What is Tyrosine?

- ▶ Non-essential amino acid (formed from essential phenylalanine)
- ▶ It plays a key role in resistance to stress!
- ▶ Tyrosine is excreted in the urine and its limit amount in a healthy adult population is max 200mg/per liter of urine  
(child under 1 year - 1500mg/L, juveniles under 18 years of age - 500mg/L)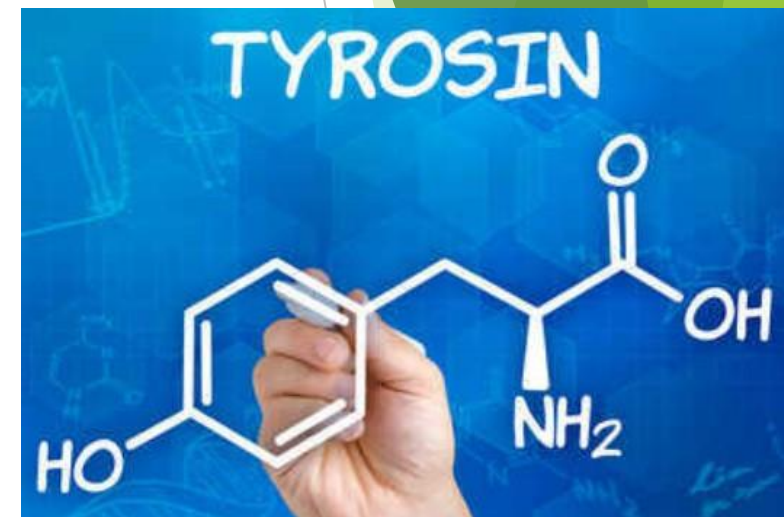

# Urinary tyrosine as a marker of malignant tumor


- ▶ High levels of tyrosine in the urine indicate a serious problem in the body (adult max. 200mg/L)
- ▶ Tyrosine plays an important role in cancer (current scientific studies)
- ▶ Urine is a medium that is ideal for non-invasive and rapid diagnosis of possible cancer

Carci Reagent

# CarciReagent for self-testing

- ▶ CarciReagent - in vitro diagnostic medical device for self-testing, intended for the detection of metabolites of monohydroxyphenol (tyrosine) and its indicative amount in the patient's urine
- ▶ The test uses a chemical chromogenic method. The idea is that the presence of a substance of interest in a test sample is detected by a chemical reaction that produces visible color changes. The result can then be compared to a color scale.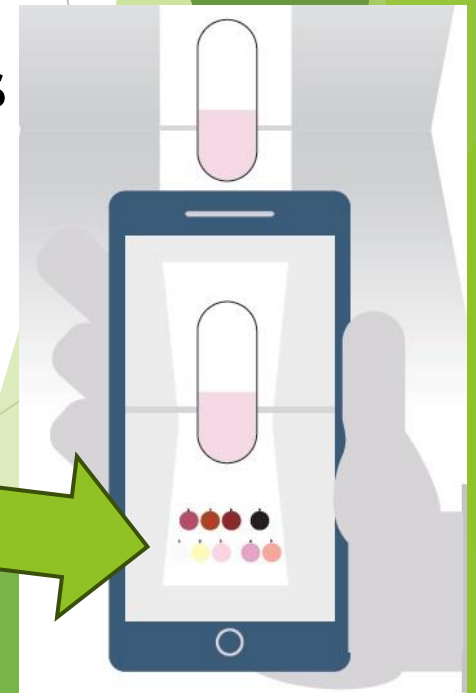

# CarciReagent for self-testing

Detailed representation (video) of the workflow:

# CarciReagent for self-testing

Rating (according to the color scale included in the package):

**Result #1, 2, 3: NEGATIVE - The amount of tyrosine in the urine is not increased**

**Result #4, 5: POSITIVE – The amount of tyrosine in the urine is increased. It is recommended to consult the result with a doctor and take an interest in your health.**

**Result No. 6, 7, 8: POSITIVE – The amount of tyrosine in the urine is significantly increased. It is recommended to see a doctor immediately and consult the test result.**

Table with approximate amount of tyrosine in urine in mg/L

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## ► Real positive test results

# CarciReagent for self-testing

Sensitivity (Malignant tumor 96.7%) and specificity (99.4%) of the CarciReagent tester

| Disease                                    | Senzitivity          |
|--------------------------------------------|----------------------|
| <b><u>Malignant tumors</u></b>             | <b><u>96,70%</u></b> |
| Pigment disorders                          | 22,10%               |
| Diabetes                                   | 21,50%               |
| Gastritis                                  | 15,10%               |
| Gastric ulcers                             | 14,10%               |
| Tuberculosis                               | 8,90%                |
| Viral hepatitis                            | 8,80%                |
| Benign prostatic hyperplasia               | 7,10%                |
| Inflammation of the gallbladder            | 7,00%                |
| Pneumonia                                  | 6,90%                |
| Esophagitis                                | 5,40%                |
| Inflammation of the small intestine        | 4,50%                |
| <b>Specificity in a healthy population</b> | <b>99,40%</b>        |

Carci Reagent

# CarciReagent for self-testing

- ▶ **THIS LAY TEST IS USED AS AN EARLY WARNING TO PREVENT POSSIBLE SERIOUS ONCOLOGICAL DISEASE!**
- ▶ **The results of the examination must be consulted with a doctor. The test is intended for single use. The test is intended for healthy persons over 18 years of age.**

# EU CERTIFICATION No. 22 0196 CN/NB

## CarciReagent IVD self-test in vitro diagnostic tool

meets the basic requirements defined in Annex I., Section A, Section B, in Annex III. § 6., directive of the European Parliament and the Council 98/79 / EC, which apply to it taking into account the intended purpose of use as defined in Government Regulation No. 56/2015 Coll.

The following requirements are met

- EN ISO 13485:2016
- EN 13532:2002
- EN 13612:2002
- EN ISO 14971:2012
- EN ISO 15223-1:2016
- EN ISO 18113-1:2011
- EN ISO 18113-4:2011
- EN ISO 23640:2015

# EU CERTIFICATION No. 22 0196 CN/NB

Carci Reagent

CNEU MEDICAL, s.r.o.  
Jirábkova 1459/8, 149 00 Prague 4, Czech Republic

## EC Declaration of Conformity

No. of IVDR: CR/901

Pursuant to Art No. 23(1)(97) CoL, On technical requirements for products, as amended, Government Decree No. 56/2015 Coll., Laying down technical requirements for medical devices for self-testing, as amended and in accordance with the requirements of the Directive 98/79 / EC of the European Parliament and of the Council of 27 October 1998 on in vitro diagnostic medical devices.

Manufacturer: CNEU MEDICAL  
Registered office: Prague 4, Jirábkova  
ID: 082 90 458  
Hereby declares that the diagnostic device

CarciReagent

meets the basic requirements defined in / Directive 98/79 / EC of the European Parliament and of the Council of 27 October 1998 on in vitro diagnostic medical devices.

Description of the notified device:  
CarciReagent - in vitro diagnostic device measuring hydroxyphenol metabolites (tyrosin) (Lys semi-quantitative test for self-testing)

These requirements are met during the period of validity of the following standards:  
EN ISO 13485:2016  
EN 13512:2002  
EN 13612:2003  
EN ISO 14971:2012  
EN ISO 13223-1:2016  
EN ISO 18113-1:2011  
EN ISO 18113-2:2011  
EN ISO 22640:2015

The notified body participated in the CE marking process.  
Notified body reference: 1023  
Notified body number: 1023  
ID: 179 0 381

ID: 082 90 458  
Bank CZK: FIO Bank; IBAN: CZ24 2010 0000 0023 0174 9129; BIC: FIOBCZPPXXX  
Bank USD: FIO Bank; IBAN: CZ49 2010 0000 0023 0174 9132; BIC: FIOBCZPPXXX  
Bank EUR: FIO Bank; IBAN: CZ39 2010 0000 0023 0174 9133; BIC: FIOBCZPPXXX  
[www.carciareagent.com](http://www.carciareagent.com)

CNEU MEDICAL, s.r.o.  
Jirábkova 1459/8, 149 00 Prague 4, Czech Republic

## EC Certificate No.: 22 0196 CN/NB

Valid from: 2022-05-20  
Valid until: 2025-05-26  
First issued: 2022-05-20

EC Certificate issued for the diagnostic device in vitro CarciReagent.

In Prague on: May 27, 2022

Responsible person: Mag. Martin Muzikant, Ph.D.

On behalf of company: CNEU MEDICAL, s.r.o., Jirábkova 1459/8, Prague 4, Czech Republic

ID: 082 90 458  
Bank CZK: FIO Bank; IBAN: CZ24 2010 0000 0023 0174 9129; BIC: FIOBCZPPXXX  
Bank USD: FIO Bank; IBAN: CZ49 2010 0000 0023 0174 9132; BIC: FIOBCZPPXXX  
Bank EUR: FIO Bank; IBAN: CZ39 2010 0000 0023 0174 9133; BIC: FIOBCZPPXXX  
[www.carciareagent.com](http://www.carciareagent.com) [info@carciareagent.com](mailto:info@carciareagent.com)

Notified Body 1023  
INSTITUTE FOR TESTING AND CERTIFICATION, Inc.,  
Jirábkova 258, Louky, 753 02 Zlín, Czech Republic

## EC Design-Examination Certificate No. 22 0196 CN/NB

Issued for manufacturer

CNEU Medical, s.r.o.

Jirábkova 1459/8, 149 00 Prague 4, Czech Republic

in accordance with requirements of

Directive 98/79/EC

on in vitro diagnostic medical devices, Annex III (6)

for the following product category(ies):

CarciReagent

The Notified Body No. 1023 has performed an examination of the design dossier relating to devices / device categories in accordance with IVDD Annex III (6). The design of the devices conforms to the requirements of this Directive.

Valid from: 2022-05-20  
Valid until: 2025-05-26  
First issued: 2022-05-20  
Revision: -

Mgr. Jiří Hoš  
Representative of the Notified Body No. 1023

Date: 2022-05-20

Page 1/1

Notified Body 1023  
INSTITUTE FOR TESTING AND CERTIFICATION, Inc.,  
Jirábkova 258, Louky, 753 02 Zlín, Czech Republic

## Design-Examination Certificate

No. 22 0196 CN/NB

Issued for manufacturer:

CNEU Medical, s.r.o.  
450/8, 149 00 Prague 4, Czech Republic

| Reference Number | Action        |
|------------------|---------------|
| 813001009        | Certification |

Mgr. Jiří Hoš  
Representative of the Notified Body No. 1023

Date: 2022-05-20  
Revision: -

Page 2/2

# REGISTRATION WITH THE STATE OFFICE FOR DRUG CONTROL, CZECH REPUBLIC, EU

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# EARLY WARNING OF SERIOUS DISEASE

- ▶ Currently, we have offered a tester to Czech health insurance companies for preventive programs within their projects.
- ▶ We are negotiating with representatives of the Czech Medical Chamber and other institutions that deal with health protection and prevention of oncological diseases

# EARLY WARNING OF SERIOUS DISEASE

- ▶ An examination is recommended at least once a year, more often in case of doubts about the state of health or other circumstances
- ▶ The recommended retail price is set at 79 USD
- ▶ You can find it on google - CarciReagent

# Thank you for your attention!

## Are you interested in providing a tester to your customers?

Do not hesitate to contact us!

Mag. Martin Muzikant, Ph.D.

Email: [muzikant@carcireagent.com](mailto:muzikant@carcireagent.com)

Mobile: +420722170087

[www.carcireagent.com](http://www.carcireagent.com)